Instructions for the EZ-Vac

The EZ-Vac is intended for the single-user to assist in drawing an adequately sized blood drop for glucose monitoring. Please read the following directions carefully before using.

Directions:

1. Select a hairless area on the forearm and avoid perforating any visible veins.
2. Clean the skin using soap and warm water.
3. If your skin is unusually dry or flaky, a small amount of handcream will help maintain the vacuum inside the EZ-Vac.
4. Make a skin puncture on your forearm using a **standard gauge** (not extra thin) lancet. Fig. 1
5. Placing your index and middle fingers under the EZ-Vac flanges, depress the plunger with your thumb. Fig. 2
6. Hold the plunger down and place the transparent tip of the EZ-Vac over the puncture site. The initial blood drop should be in the approximate center of the EZ-Vac tip. Fig. 3
7. Release the plunger. The blood drop will slowly enlarge over the next several seconds. Fig. 4
8. Repeat steps 5-7 a second time if necessary to further enlarge the blood drop. Fig. 5
9. Depress the plunger to release the EZ-Vac from the skin.
10. Apply the blood drop on a test strip and read the result on your glucose meter. Fig. 6

Note: EZ-Vac is best used with test strips that require 3 µL or less of blood (Examples: Bayer Glucometer Elite and Lifescan Fast Take TM)

Taking care of your EZ-Vac:

Use Alcohol or soap and water to wipe the inside and outside of the EZ-Vac. For more complete cleaning, wash it in warm water and rinse well. It can be disinfected with 70% rubbing alcohol (hospitals should consult their own infection control protocol).

Limited Warranty:

AMBIMEDINC warrants that the EZ-Vac will function effectively for a period of one (1) year from the date of purchase. A malfunctioning device should be returned to your supplier or to AMBIMEDINC for replacement.

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