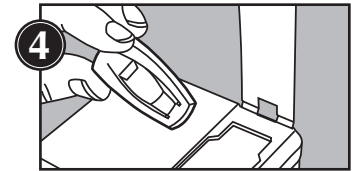
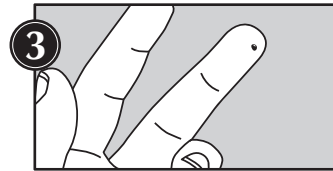
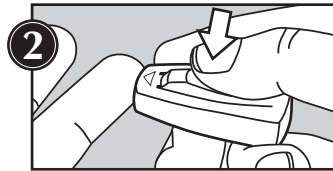
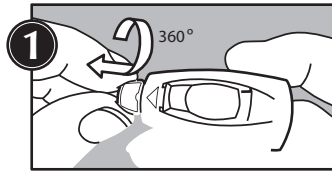
# EZ-Lance®

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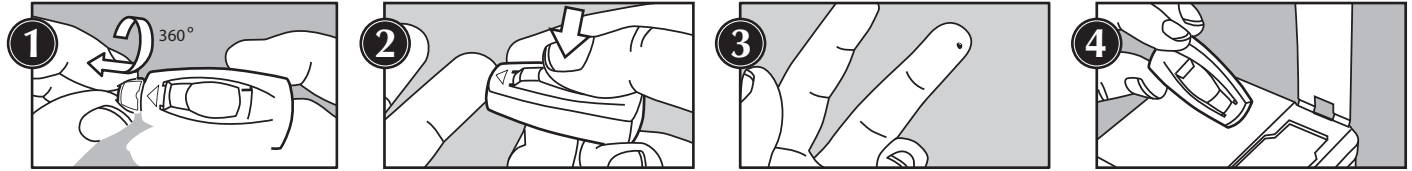
# EZ-Lance®

## Instructions for the EZ-Lance®

The EZ-Lance is a single-use, disposable lancing device that automatically injects the lancet into and retracts it from the fingertip to obtain a capillary blood sample for glucose monitoring or other laboratory blood tests.

### CAUTIONS:

- The EZ-Lance is for a single use only
- Do not use the lancet if the cap (see Figure A) has been removed or is missing
- Do not use the lancet if the indicator bar is broken (see Figure B-1)
- The EZ-Lance contains small parts and could be a choking hazard for children

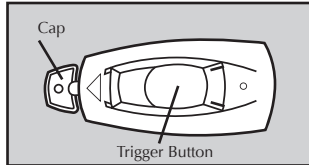
Figure A - Top

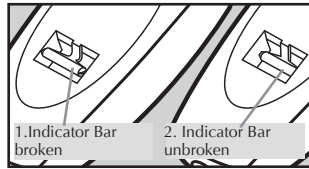
Figure B - Bottom

### Instructions for use:

Clean hands with soap and warm water; dry them thoroughly. Warming hands will help in obtaining a blood sample. Follow the steps below. Each step corresponds to the numbered-illustrations.

1. Hold device as indicated in illustration 1, remove cap by twisting and then pulling it off.
2. Hold the device as indicated in illustration 2 with thumb on the trigger button. Place the end of the EZ-Lance marked with the arrow against the pad of the finger to be lanced. The best puncture sites are on the middle or ring fingers. Push the trigger button firmly to activate the device. The lancet automatically retracts after puncturing the site.
3. Set aside the EZ-Lance and wait a few seconds for a blood drop to form. The flow of blood will be helped by keeping the hand warm, by lowering the hand to waist level, and by gentle massage of the finger.
4. Dispose of the EZ-Lance immediately after use in an appropriate sharps or biohazard container.

### Storage requirements:

To maintain product function and integrity, protect from moisture and extremes of temperature. Do not store in direct sunlight.

### Warranty:

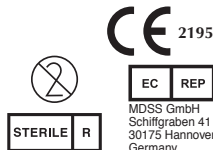
This AMBIMEDINC product carries a one (1) year warranty against manufacturer's defects in materials and workmanship and against breakage or failure under normal use. This warranty does not cover misuse or abuse. AMBIMEDINC will, at its option, issue credit or replace this product at no charge. This warranty gives you specific legal rights, and you may also have other rights under law.

EZ-Lance is a registered trademark of AMBI, INC. • US Patent #7,282,058 • US Design Patent #D499,182 • International Patents pending • All rights reserved

Made in USA

**AMBIMEDINC**  
World Class Healthcare Products

908 Sir Francis Avenue • Capitola, CA 95010, USA  
Phone/Fax 1-831-475-1765  
www.ambimedinc.com • info@ambimedinc.com

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