

# Insul-Cap<sup>®</sup>

---

## Frequently Asked Questions/ Troubleshooting



**Q: What size syringe works with the Insul-Cap?**

A: The Insul-Cap works with most 1ml, 1/2ml and 1/3ml (100 units, 50 units and 30 units) insulin syringes with 6mm (15/64") or longer needle length. The Insul-Cap also works with 1ml tuberculin syringes if they have a fixed needle or Leur-Slip Needle.

**Q: What type syringe does not work with the Insul-Cap?**

A: The Insul-Cap does not work with Leur-Lock syringes or 3ml or larger syringes.

**Q: Do the Insul-Caps work with syringes used on infusion pumps?**

A: The Insul-Cap works with the Omnipod syringe that is used to load the insulin into the Omnipod insulin pump.

**Q: What size vial works with the Insul-Cap?**

A: The Insul-Cap fits Eli Lilly\*, Novo Nordis and Aventis Lantus\* insulin bottles.

**Q: Do the short BD\* syringes work with the Insul-Cap?**

A: Yes, the Insul-Cap works with the short syringes as long as the needle length is 6mm or longer.

**Q: Does the Insul-Cap contain latex?**

A: The Insul-Cap does not contain latex. For a list of materials please contact AMBIMEDINC.

**Q: How do I clean the Insul-Cap?**

A: Submerge the Insul-Cap in a soap and water solution, rinse and air dry.

