

# ambi-Lancet<sup>®</sup> Standard, Mini and Eject

## Frequently Asked Questions/ Troubleshooting

**Q: What lancets other than EZ-Lets<sup>®</sup> are compatible for use with the ambi-Lancet<sup>®</sup> ?**

A: The following lancets are compatible with the ambi-Lancet

- Microlet\* [Bayer]
- Omnican\* [BBraun]
- CleanLet\* [Facet Technologies]
- Ultra TLC\* [Abbott]
- One Touch\* [Lifescan]
- Unilet GP\* [Owen Mumford]
- Freestyle\* [Therasense]
- Qualitilet\* [Facet Technologies]
- Monolet\* [Sherwood]
- UltraFine\* II [BD]
- SoftTouch\* [Roche Australia]
- SoftTouch\* [Roche Accu-Chek]
- Gentle-Let\* [Ulster Scientific]

**Q: What lancets are not compatible for use with the ambi-Lancet<sup>®</sup> ?**

A: The following lancets are not compatible for use with the ambi-Lancet<sup>®</sup> :

- TechLet\* [Diamed]
- SureLite\* XL [Facet Technologies]
- Greenlan\* [DukNam Med]
- UltiCare\* [UltiCare]
- SureLite\* [Facet Technologies]
- Softclix\* [Roche]

**Q: Which depth of penetration adjustment should I use?**

A: Choose settings #1 or #2 for soft, thinner skin for a sufficient blood drop, or on average skin for a smaller blood drop. Choose setting #3 for average skin (not too thin or thick) to obtain a sufficient blood drop, or settings #4 or #5 for thicker, calloused skin or for a larger blood sample size requirement.

**Q: Can the ambi-Lancet<sup>®</sup> be used on the forearm?**

A: The ambi-Lancet<sup>®</sup> alone is not intended to be used on the forearm because the adjustable tip is not designed to properly coax a blood drop from the forearm. The ambi-Lancet<sup>®</sup> in conjunction with the EZ-Vac<sup>®</sup>, however, can produce an adequately sized blood drop from the forearm.

**Q: How do I clean the device after use?**

A: Use 70% rubbing alcohol or soap and water to wipe the outside of the lancing device. For more complete cleaning, remove the tip, wash it in warm water and rinse well. Do not clean the ambi-Lancet<sup>®</sup> in the dishwasher.

**Q: How do I safely dispose of a used lancet when using the ambi-Lancet standard or ambi-Lancet mini?**

A: In order to avoid an accidental scratch or puncture, make sure that you reapply the safety cap of the disposable lancet without picking it up with your fingers. You can do this by resting the cap on a surface and by pressing the used needle against it. When the cap securely covers the used lancet, remove it with your fingers and dispose in a proper biohazard container.

**Q: How do I safely dispose of a used lancet when using the ambi-Lancet Eject?**

A: Press lancet ejector button at the same time you pull back the lancing device cocking mechanism. Place the used lancet into an appropriate biohazard container.

**Q: Can the ambi-Lancet<sup>®</sup> be used on more than one person?**

A: The ambi-Lancet<sup>®</sup> is intended for the single user. To avoid exposure to contaminants, do not use the ambi-Lancet<sup>®</sup> on multiple users.

**Q: Does the ambi-Lancet<sup>®</sup> contain latex?**

A: The ambi-Lancet<sup>®</sup> does not contain latex. For a list of materials please contact AMBIMEDINC.

