

HAN-Ease (Hide-A-Needle)

The HAN-Ease attaches to Pen-Injectors to conceal the needle, helping to ease the pain and anxiety of injections

Frequently Asked Questions/Troubleshooting



Q: What Pen-Injector works with the HAN-Ease?

A: Most Pen Injector with needles 4mm to 12mm work with the **HAN-Ease**.

Q: How do I clean the HAN-Ease after use?

A: The HAN-Ease can be washed with warm soapy water and air dried.

Q: Can the HAN-Ease be used on more than one person?

A: The HAN-Ease is intended for the single user. To avoid exposure to contaminants, do not use the HAN-Ease on multiple users.

Q: Does the HAN-Ease contain latex?

A: The HAN-Ease does not contain latex. For a list of materials please contact AmbiMedInc.