


Instructions for Insul-eze® SPM

Pen-Injector Dial and Large Syringe Magnifier

The Insul-eze SPM (IZ SPM) is a small, easy to clip-on magnifier for most pen-injector dials and the barrel of large 3ml, 5ml and 10ml syringes. The magnifier helps to easily see the medication level by increasing the viewing area of a pen-injector dial or the syringe markings. The easy to squeeze mechanism makes it simple to put on and take off.



Insul-eze SPM with Pen-Injector

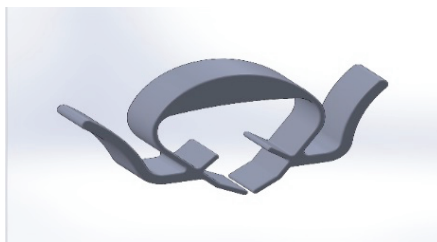


Insul-eze SPM



Insul-eze SPM with 3ml syringe

Directions:


After following the directions from your Pen-Injector or Syringe Manufacturer and the site preparation from your Medical Care Team, continue with the following Insul-eze SPM (IZ SPM) instructions:

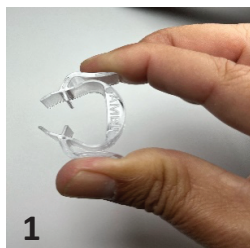

1. Place forefinger and middle finger on each side of wings of the IZ SPM.



2. FOR PEN-INJECTORS: Squeeze the wings of the IZ SPM so it opens large enough to allow the pen-injector dial to fit under the magnifier. Once you have dialed the desired amount, remove the IZ SPM by squeezing the wings. Continue with the Pen Manufacturer's and the Health Care Team's instructions for use.

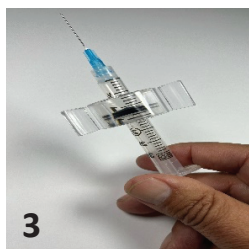

3. FOR LARGE 3ml, 5ml, and 10ml SYRINGES: Squeeze the wings of the IZ SPM so it opens large enough to allow the large syringe to fit under the magnifier. Once you have the desired medication amount, remove the IZ SPM by squeezing the wings. Continue with your Health Care Team's instructions for use.



Capitola, CA 95010 USA • 831-475-1765

www.AmbiMedInc.com • info@AmbiMedInc.com

230-30-2031SPM